




Instruction for Use -

Assembly, Installation and Operation of the S0209 Foot Operated Soap Pump Assembly Kit (for Scrub Sinks)

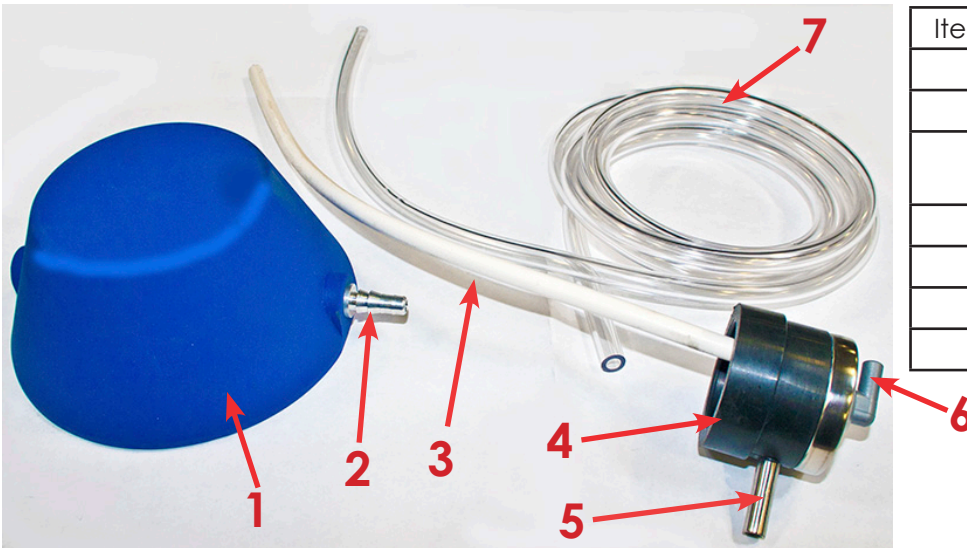

Item #	Description
1	Foot Pedal
2	Foot Pedal Nozzle
3	Soap Cap Suction Tube (pre-attached to the Soap Cap)
4	Soap Cap
5	Soap Cap Side Nozzle
6	Soap Cap Top Nozzle
7	Transfer Tubing 6 FT

1. Cut the Transfer Tubing (Item 7) in two sections, a short section to connect from the Foot Pedal (Item 1) to the Soap Cap (Item 4); a longer section to connect from the Soap Cap to the Soap Pump Spout mounted on the top of the sink.
2. Push one end of the shorter Transfer Tubing onto the Foot Pedal Nozzle (Item 2).
3. Push the other end of the shorter Transfer Tubing onto the Soap Cap Side Nozzle (Item 5) of the Soap Cap (Item 4).
4. Push one end of the longer Transfer Tubing onto the Soap Cap Top Nozzle (Item 6).
5. Push the other end of the longer Transfer Tubing onto the Scrub Sink Soap Pump Spout.
6. Ensure all tubing is pushed onto each of the nozzles as far as it will go to prevent leaking.
7. Remove the soap bottle cap and replace it with the Soap Cap (Item 4). Make sure to insert the Soap Cap Suction Tube (which protrudes from the Soap Cap) into the soap bottle. Ensure the soap cap is pushed down onto the soap bottle securely.
8. To operate, repeatedly depress the Foot Pedal to pump the soap to the sink Soap Pump Spout.

Foot Operated Soap Pump Assembled

