


Instruction for Use - Cleaning the Soap Pump and Tubing for a MAC Medical Scrub Sink

These instructions cover the cleaning of a MAC Medical Soap Pump and its associated tubing.

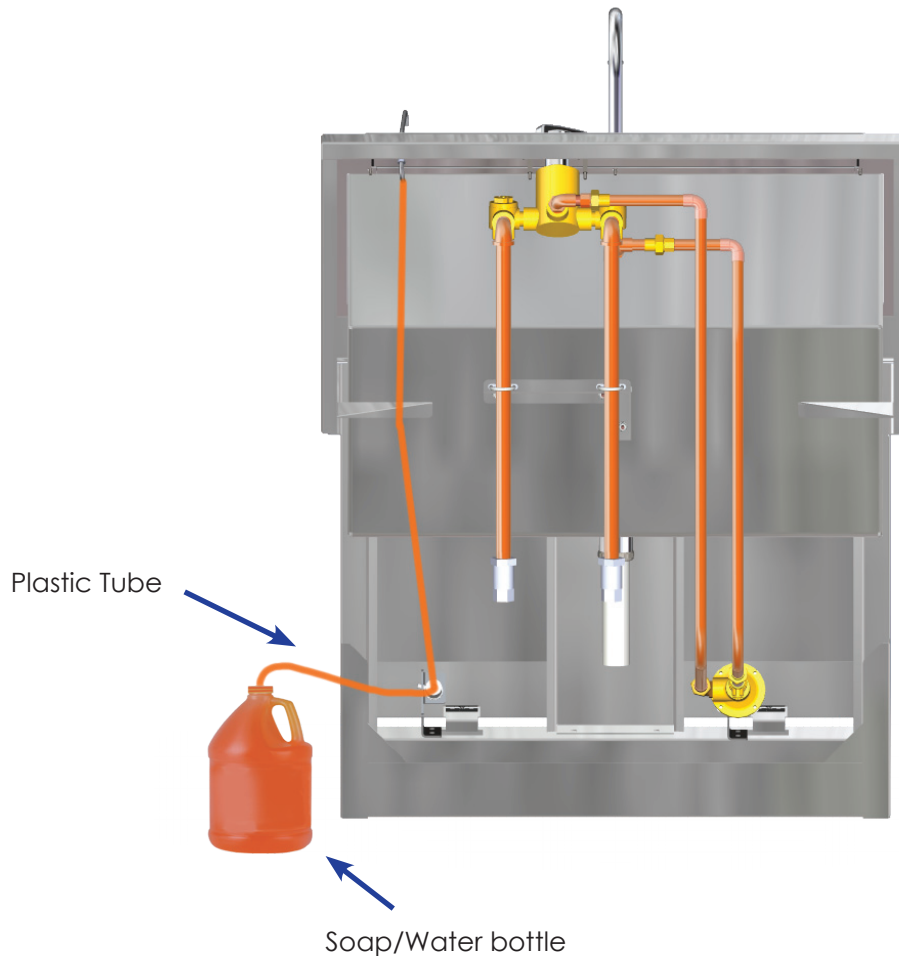

1. Disconnect the current soap bottle (if one is present) from the system by removing the plastic tubing from the bottle.
2. Operate the pump to dispense any soap remaining in the system. Continue to operate the pump until no more soap comes out.
3. Fill an empty bottle with clean, warm water. You may add a teaspoon of vinegar to the water to remove any soap clogs that might be present in the system and can serve as a safe disinfectant.
4. Connect the plastic feeder tube to the bottle of water as you would do with the soap bottle. Operate the pump as normal and allow the clean water to go through the system. Continue to pump until the water flows and pumps freely.
5. Remove the tube from the bottle of water. Discard any remaining water in the water jug. Connect the plastic feeder tube into the bottle of soap.
6. Operate the pump as normal until soap begins to flow through the system.