




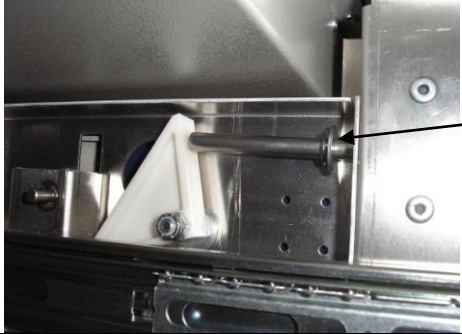
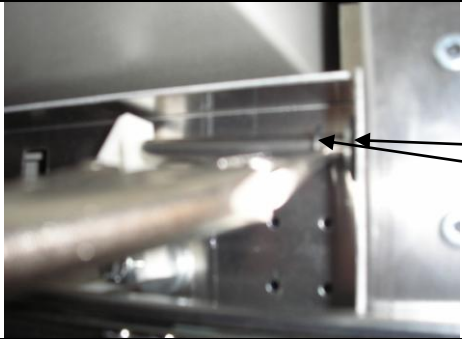





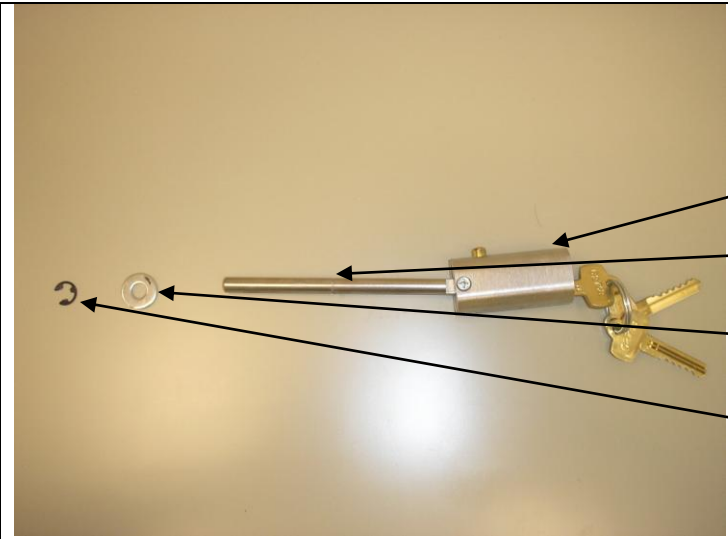



AL/SL BEST LOCK REPLACEMENT

		<p>Best lock replacement for AL and SL series carts.</p>
		<p>Unlock cart.</p> <p>Cart shown locked.</p>
		<p>Cart unlocked.</p>
		<p>Fully open the top drawer.</p>
	<p>Remove the drawer from the cart.</p> <p>The drawer releases are located in the center of the drawer member of the slide as shown.</p> <p>Left side lift, right side push down.</p>	

AL/SL BEST LOCK REPLACEMENT

	<p>With the drawer removed, push cart lock in to expose the E-clip and washer.</p>
	<p>Slide flat washer forward to get access to E-clip.</p> <p>Flat washer</p> <p>E-clip</p> <p>Using needle nose type</p>
	<p>E-clip removed.</p>
	<p>Unlock the cart and start to slide the lock out the front of the cart.</p>
	<p>As the lock is sliding out the cart, reach in and get the flat washer before it falls down into the cart.</p>

AL/SL BEST LOCK REPLACEMENT

		<p>The components of the lock system.</p> <p>Best lock w/3 keys 640681</p> <p>Lock rod 34000046</p> <p>Flat washer 1/4" 45250301</p> <p>E-clip 5/16" 45250226</p>
		<p>The new lock comes with the screw for the lock rod. Remove the lock rod off the old lock and replace on the new lock.</p>
		<p>With the new assembly in hand slide the lock rod through the front of cart.</p>
		<p>Before fully installing add the flat washer to the lock rod on the inside.</p>

AL/SL BEST LOCK REPLACEMENT

		<p>Make sure the cam is as shown.</p> <p>Cam</p>
		<p>Push the lock in and lock the cart.</p> <p>Slide the flat washer all the way forward.</p> <p>Add the E-clip to the lock rod in the groove.</p>
		<p>Check the lock.</p> <p>Unlock and relock a few times.</p>
		<p>Fully extend the cabinet members.</p>
		<p>Reinsert the drawer into the cabinet members of the cart. Full close and open a few times. Lock the cart and test.</p>