



The Harloff Company, Inc.
650 Ford Street
Colorado Springs, CO 80915-3712
(800) 433-4064 || (719) 637-0300
Fax (719) 597-8273

Lockslide Adjustment


Date 2021/06/10

By: Joseph Perez

Tools Needed – Flat Tip Screwdriver

1. Remove all drawers and lock the cart
2. Front right of the cart is the lockbar. The tabs on this should **not** rest on the slides. Also, if the drawers are not unlocking or getting stuck in the unlocked position then the lockbar needs to be adjusted.

Correct Position of tabs on lockbar

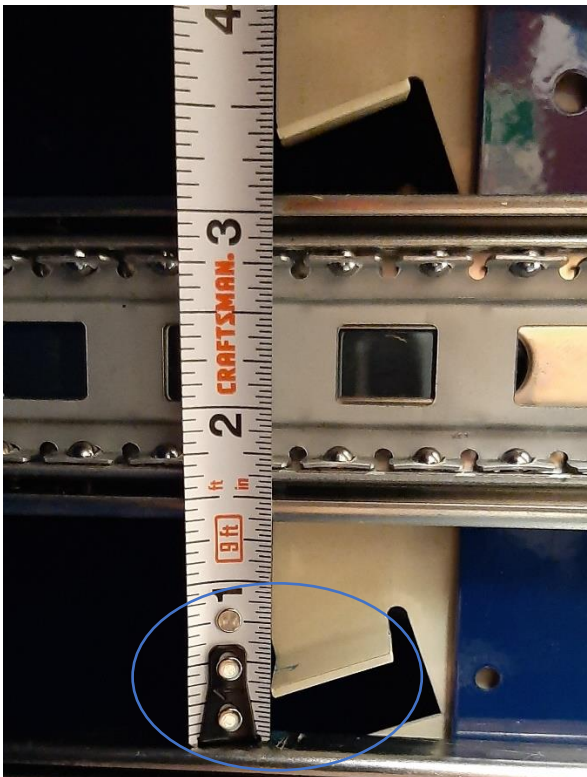

3. To adjust the lockbar use a flat tip screwdriver putting it between the top of the lockbar and the top of the cart and pull the lockbar up with your hand.



4. The lockbar should be bent at a 90-degree angle (you can use the ceiling of the cart as a reference).
5. You may have to slightly overbend the lockbar for the drawers to unlock.
6. If the lockbar is overbent too far and the drawers don't lock it can be underbent. With the lockbar resting on the plastic cam slightly pull down on the lockbar.



7. Put a drawer in the cart and test the drawer ensuring it locks and unlocks correctly.
8. Once you confirmed the drawers are locking and unlocking correctly keep the cart unlocked and add all the drawers back to the cart. The cart can now be used as normal.