



Instruction Sheet VL-Key Lock Plate



* Check to see if all parts are there



1. Take out drawers
2. Drill Out Rivets holding existing plate



3. Remove Lock and plate



4. Hold lock so key is upright with teeth of key pointing upwards. (Key must turn to right and stop)



5. Slide lock into plate



6. Add clip to back of plate (sideways)



7. Turn key to left, with teeth down



8. Add plastic white cam, with larger portion/end down
9. Add screw to back of lock and tighte

Instruction Sheet VL-Key Lock Plate



10. Pull key out and put plate with lock in place in the cart's frame



11. Raise lock bar up so plastic cam meets up with horizontal extension of lock bar.



- 12. Put new rivets in place and install with rivet gun
- 13. Put key in place and check that it moves the lock bar
- 14. Re-install drawers